



For Dogs and Cats.

**Episanis™ BioHance™ Skin and Wound Gel** is formulated to support an enhanced healing process of skin from acute injuries, surgical incisions and common skin irritations in dogs and cats. It also promotes rapid wound closure. BioHance is a unique, crosslinked hyaluronic acid (HA) that has been specifically modified for the epidermal environment of dogs and cats. It creates a cellular matrix of modified HA that naturally enhances the animal's own healing process, facilitates epithelial healing and helps reduce scarring. The soothing gel provides a thin coating on the surface of the skin that helps protect the affected area to help reduce risk of infection.

### Indications

For use on superficial wounds (cuts, scratches and abrasions), traumatic wounds (bites, lacerations and punctures), surgical incisions, hot spots and skin ulcers.

### Effectiveness

The efficacy of this product is supported with research conducted by a major university in the United States. The patented BioHance modified hyaluronic acid crosslinking technology has been shown to help accelerate the rate of healing in acute injuries to the skin.

### Ingredients

All ingredients are naturally occurring in the body and do not cause pain or stinging.

- Water
- BioHance Hyaluronic Acid (1.0%)
- BioSalts
  - Sodium Chloride
  - Disodium Phosphate
  - Potassium Chloride
  - Potassium Phosphate

Contains no preservatives, antibiotics or steroids.

**Warning:** For animal use only.

### Adverse Reactions

For product support or to report an adverse reaction call 833-224-2015.

### Administration

1. Wear gloves to assure sterility and cleanliness.
2. Properly clean the affected site where the spray gel will be applied.
3. Spray Episanis BioHance Dermal Repair Gel to completely cover affected area.
4. After spray material is in place, the site can be bandaged as directed by your veterinarian or per normal treatment of the specific type of skin injury, incision or irritation.
5. A single daily application may be sufficient to help enhance healing and closure in fresh, uncomplicated wounds. More complex injuries, irritations or incisions may require further treatment to accelerate closure. Consult your veterinarian.
6. To avoid contamination, do not touch the tip of container to any surface.
7. Replace cap securely after using.

### Storage

Product should be stored at room temperature, not above 85°F (29°C). May be stored in the refrigerator. If refrigerated, the product should be allowed to warm to room temperature prior to use. **DO NOT FREEZE.** Store upright.

### How Supplied

15mL spray-dispenser bottle

For customer service, contact SentrX™ at 833-224-2015.

Manufactured by  
SentrX Animal Care, Inc.  
Salt Lake City, UT 84108

